

# HEINE HR®/HRP®/HR-C®/ Binokularlupen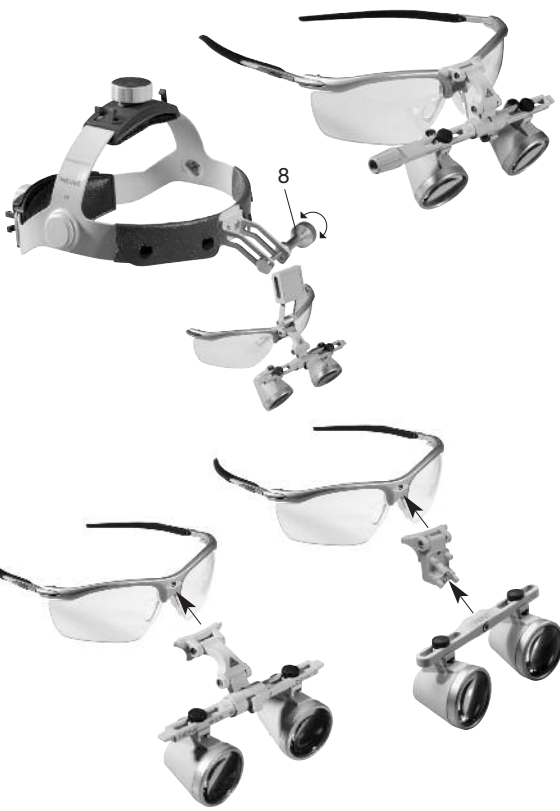

CE 93/42/EWG/CEE/EEC

## HEINE

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49(0)8152/38-0 · Fax +49(0)8152/38-202  
E-mail: info@heine.com · www.heine.com  
med 2606 1/2.10

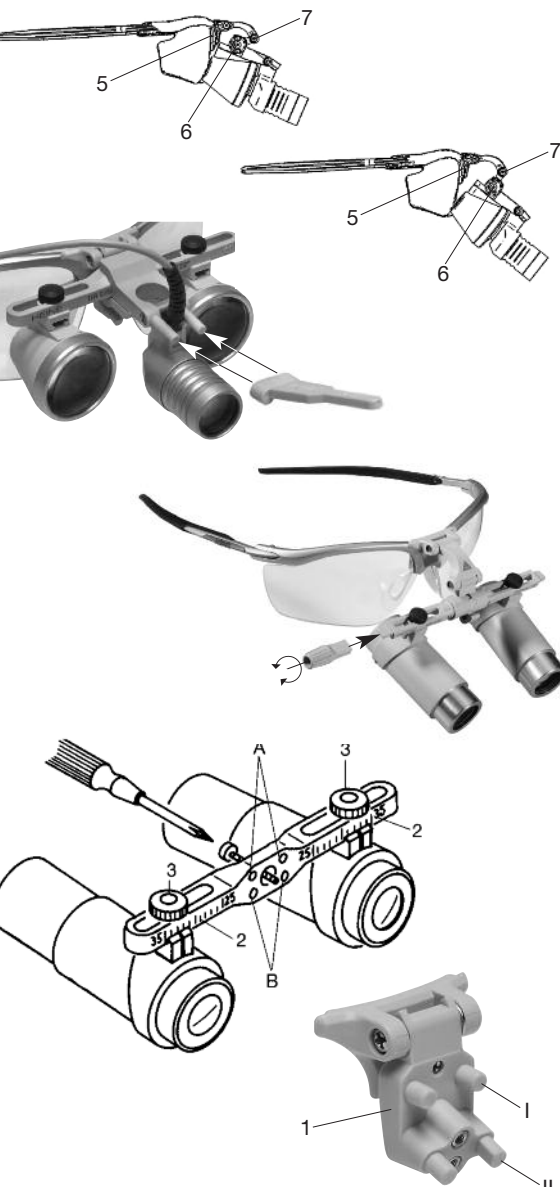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DEUTSCH

### HEINE HR®/HRP®/HR-C® Binokularlupen

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ENGLISH

### HEINE HR®/HRP®/HR-C® Binocular Loupes

Before using your HEINE HR®/HRP®/HR-C® Binocular Loupes read these instructions carefully and keep them in a safe place for future reference.

#### Safety information

**Range of applications:** HEINE HR® Loupes are designed only for optical magnification during examinations and operations.  
**Safety in use:** Do not look directly at the sun or into a powerful light source with the loupes to avoid the possibility of eye damage.  
The angle of convergence of the eyepieces was set by the manufacturer by means of the screw 4. This setting may only be adjusted by a competent technician.

#### Assembly and use

**How to fit HEINE binocular loupes with HEINE i-View® to the HEINE S-Frame®:**

The HEINE i-View® allows you to select the angle of view of the loupes individually. This is done by means of a triple link system (5, 6, 7).

#### Swivelling the loupes:

The HEINE i-View® can be swivelled by hand as shown above. To do so, attach a start-lever to one end of the optics mount. The loupes can be moved out of the line of sight by rotating the lever. This allows you to view the object without magnification and avoids the need to re-adjust the light.

**How to fit HEINE binocular loupes with HEINE i-View® to the Professional L Headband with S-Guard splash protection**

Unscrew the fixing screw (8) and the S-Guard can be moved vertically and horizontally. Fix the loupes with the fixing screw (8) as shown in Fig. in the desired position.

**How to attach HEINE binocular loupes without HEINE i-View® to the HEINE S-Frame®**

HEINE HR® Loupes can be fixed to the frame or other carrier system in three different positions:

B-I: horizontal B-II: angled slightly down A-II: as in B-II, but mounted 5 mm lower

Your loupe is fitted to one of the following carrier systems. The swivel attachment can be fitted to both the S-Frame and the Combi frame.

**Professional Headband:** Circumference and height can be adjusted by means of the two controls.

**MD 1000 F.O. and 3S LED HeadLight:** Attach the HEINE binocular loupes by means of the loupe adaptor or HEINE S-Guard® to the HeadLight. To do so, please consult the instructions supplied with the HEINE S-Guard®  
**Combi Frame:** Adjust the length of the temples to suit your needs.

#### Adjustment

The HR Loupe should be adjusted very carefully to assure the best possible view.

To begin **setting up proceed** as follows:

- Select the inter-pupillary distance (PD) or an average value on the scale (2): to do this, loosen the screws (3) and slide both eyepieces in or out until they are symmetrical.
  - Now try on the **HEINE S-Frame®** or **headband** and make sure that the eyepieces are **horizontal and symmetrically-placed in front of your eyes**.
  - Now look at a well-lit, three-dimensional object (not a text or similar) at the **normal working distance** and adjust the eyepieces so that you can see the object with both eyes in the middle of the field of view.
  - **Horizontally** – by varying the position of the eyepieces or, with the headband, by pushing it to one side or the other.
  - **Vertically** – by swivelling the adaptor or after having removed the loupe by selecting a different height adjustment.
- The loupe is correctly adjusted when you see a round, shadow-free image.**
- Fix the eyepieces in place by re-tightening the screws (3).

#### Accessories

**HEINE binocular loupes without HEINE i-View®:**

- The **angled adaptor** is a screw-fit to the lowest setting on the mount and angles the eye-pieces to be angled downwards. This allows the operator to keep a more level head position even when looking down to operate close to his own body (does not apply to the MD 1000 F.O.).
- The washable **lens protectors** are push-fit to the eyepieces.
- The optional lights for the binocular loupes (LED LoupeLight) are described in the appropriate instructions.
- **Only for HR Binocular loupes.**
- The **close-up lenses** shorten the working distance from 340 to 250 mm and are a simple push-on fit.

#### Cleaning Instructions

**Cleaning the loupes:**

The optics of the HEINE HR® Loupes are **dust-protected and water-resistant**. They can be washed under running lukewarm water with a soft brush. Before washing, remove the close-up lenses or lens protectors. Use a micro-fiber cleaning cloth to remove fingerprints or dust from the glass lenses. All other surfaces can be cleaned with a cloth which has been moistened with a little alcohol or non-aggressive cleaner.

#### Cleaning the loupe mount HEINE i-View®

Use a micro-fiber cloth with a little alcohol or non-aggressive solution to clean the loupe mount.

#### Accessories Spare Parts

S-Frame	C-000.32.302
Only for HR/HRP Binocular loupes	
Professional headband HR/HRP	C-000.32.549
Detachable lens protectors (HR)	C-000.32.537
Detachable lens protectors (HRP)	C-000.32.027
Angled adaptors HR/HRP	C-000.32.528
Sterilisable swivel levers	C-000.32.551
Only for HR Binocular loupes	
Close-up lenses 340/250 mm HR 2x	C-000.32.521
Close-up lenses 340/250 mm HR 2,5x	C-000.32.523

#### Technical specification

The values for magnification and working distance marked on the loupes refer to normally-sighted or optically-corrected eyes ignoring accommodation. These values can vary depending on refractive error and accommodation. In general, if the examiner is short-sighted or tends to accommodate the actual working distance will be shorter. If long-sighted, it will be longer. If the working distance is shorter, the magnification will be stronger and vice versa.

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#### Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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