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GB Foot Warmer HDF

FR Chauffe-pieds HDF

IT Scaldapiedi HDF

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NL Voetwarmer HDF

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Thank you
very much

Thank you for your confidence in us and congratulations!

You have purchased the Foot Warmer **HDF**, a quality product by **MEDISANA**.

To ensure the best results and long-term satisfaction with your **MEDISANA** Foot Warmer **HDF**, we recommend that you read the following operating and maintenance instructions carefully.

IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

You will find the following symbols in the operating manual:

This instruction manual belongs to this device. It contains important information about starting up and operation.

Read the instruction manual thoroughly.

Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Do not puncture the foot warmer.

Do not use the foot warmer when it is folded.

IPX1

Drip-proof.

Only use the foot warmer indoors.

Do not chlorinate!

Do not iron the foot warmer.
The covers can be ironed at a low temperature.

Do not dry clean!

Do not dry the foot warmer in a tumble drier.
The covers can be dried in a drier.

Do not wash the foot warmer.
The covers can be washed in a normal wash at
max. 40 °C.

1.1 Safety Information

- Carefully check the foot warmer before each use for signs of wear and/or damage.
- Do **not** use if you notice wear, damage or signs of improper use on the foot warmer, the switch or the cables. Return it to the supplier.
- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- The appliance may not be switched on using a clock timer.
- The foot warmer may not be operated while folded or creased.
- No safety pins or other pointed or sharp objects may be attached to the foot warmer or stuck into it.
- The unit is designed for private home use only.
- This foot warmer is not intended for clinical or commercial use.
- Only use the foot warmer for the purpose described in these operating instructions.
- Do not use the foot warmer for small children, sick or disabled persons or any person who is non-sensitive to heat. Persons that can only feel heat up to a certain temperature may only use the foot warmer if supervised.
- The appliance may not be used by children unless the appliance has been set by a parent or person supervising, or if the child has been instructed on how to use the appliance safely.
- Children must be supervised when using the foot warmer to ensure that they do not play with it.
- The appliance is not to be used for warming animals
- Never leave the foot warmer unattended during operation.
- Do not fall asleep while the foot warmer is switched on.
- Do not wear shoes when using the foot warmer because this will lower your heat-sensitivity and will cause the foot warmer to become dirty.
- Prolonged use of the foot warmer at a high setting could lead to skin burns.
- Do not use the foot warmer in bed.
- Do not use the foot warmer when it is damp or in moist environments. The foot warmer may only be used again once it is completely dry.
- The controls and leads must not be exposed to any kind of moisture.
- Do not reach to retrieve a foot warmer that has fallen into water. Unplug the appliance immediately.
- During operation, the switch and lead may not be placed on or under the foot warmer or covered in any other way whatsoever.
- Do not place any heat sources (e.g. hot-water bottles or heating pads) on the foot warmer.

1.1 Safety Information

- Keep the mains lead away from hot surfaces.
- Do not place any large or heavy objects (e.g. cases, etc.) on the foot warmer regardless of whether the appliance is switched on or off.
- Never carry, pull or turn the foot warmer using the power cable and do not clamp the cable.
- When the appliance is not in use, disconnect the plug from the mains.
- In the event of a defect, do not repair the foot warmer yourself. Repairs must only be carried out by an authorized specialist dealer or other suitably qualified personnel.
- A damaged power supply cable cannot be replaced. If the power supply cable is damaged, the whole foot warmer must be disposed of.
- When storing the foot warmer, leave it to cool down first before folding. To prevent creases, do not place any objects on the foot warmer when storing.
- If you have any health concerns, consult your doctor before using the foot warmer.
- Do not use the foot warmer if your feet are swollen, inflamed or injured.
- If you experience any pain or discomfort while using the unit, stop using it immediately.

2 Operating

2.1 Items supplied and packaging

Please check first of all that the unit is complete and is not damaged in any way. In in doubt, do not use the appliance. Send it to a service point.

The following parts are included as standard:

- 1 **MEDISANA** foot warmer **HDF**
with insert cover (interior) and protective cover (exterior)
- 1 Instructions for use

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

Please ensure that polybag packing is kept away from reach of children! Risk of suffocation!

2.2 Operation

Always make sure that the foot warmer is not creased or folded while it is in use. You must always use the foot warmer with both covers.

Switch the appliance on with the sliding switch on the control unit. Never use a clock timer!

Switch level **0** = Appliance switched off

Switch level **1** = Heat low

Switch level **2** = Heat medium

Switch level **3** = Heat high

While the foot warmer is heating up, the indicator light is lit. As soon as the indicator light goes out, this means that the heat level set has been reached and will be maintained.

You can now enjoy the pleasant warmth at your feet. After approx. 120 minutes the foot warmer automatically switches off. To switch the foot warmer on again, you must first move the sliding switch to position **0**. After approx. 5 seconds you can switch the foot warmer back on by sliding the switch to level **1**, **2** or **3**. You can stop using the appliance at any time by moving the sliding switch to position **0**. Disconnect the plug from the mains socket when you are not using the foot warmer.

WARNING

When the foot warmer is in operation, the control unit may warm up slightly. For this reason, the control unit must not be covered or placed on the foot warmer while the appliance is switched on.

Do not rely on your hands to test the heat of the foot warmer. Hands are considerably less sensitive towards heat than the rest of the body's surface. A temperature which is judged by the hand to be slightly warm can be felt by the feet to be unpleasantly hot.

The indicator light also goes out if the safety system switches off the foot warmer due to a fault, even if the foot warmer is still switched on. The foot warmer must no longer be used after a fault has occurred. This is because safe operation of the appliance can no longer be guaranteed.

**3.1
Cleaning and maintenance**

Foot warmer

The electric foot warmer should be kept clean because it may not be washed or dry-cleaned. Before cleaning the foot warmer, always disconnect the plug from the mains and allow it to cool down. The foot warmer can be cleaned dry with a soft brush. Small marks can be removed with a cloth or a slightly damp sponge, using a mild liquid cleaner. The foot warmer may not become wet. If it becomes damp it must be air-dried. Store the foot warmer in a dry place and do not place any objects on top of it.

Insert cover (interior) / Protective cover (exterior)

Before cleaning the foot warmer, the insert cover must be taken out and the protective cover removed by undoing the velcro fasteners. Both covers can be cleaned with a damp cloth or according to the care instructions on the labels. Use solvent-free cleaners only. Only use the foot warmer again once it and both the covers are completely dry.

**3.2
Disposal**

This product must not be disposed together with the domestic waste.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Consult your municipal authority or your dealer for information about disposal.

**3.3
Specifications**

Name and model	: MEDISANA foot warmer HDF
Power supply	: 220-240 V AC 50 Hz
Heat output	: max. 60 W
Automatic switch off	: after approx. 120 min
Switching levels	: 0 - 1 - 2 - 3
Protection class II	:
Dimensions	: approx. 32 (l) x 31.5 (w) x 22 (h) cm
Weight	: approx. 650 g
Mains lead length	: approx. 2.6 m
Article No.	: 60250
EAN Code	: 40 15588 60250 4

In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.

4.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA AG

Itterpark 7-9

40724 Hilden

Germany

Tel.: +49 (0) 2103 / 2007-60

Fax: +49 (0) 2103 / 2007-626

eMail: info@medisana.de

Internet: www.medisana.de

The service centre address is shown on the last page.

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D&L Marketing Oy
ALERE Electronics AB